

Sage PrimaFit® External Urine Management for the Female Anatomy



Competency skill checklist

Name:
Date:

Placement	Completed	Comments
1. Sets up suction canister with tubing per hospital protocol.		
2. Places patient in supine position and separates patient's legs.		
3. Cleans patient's genital area from front to back per hospital protocol and assesses skin integrity.		
4. Removes device from package.		
5. Attaches suction tubing. Ensures connection to device is secure.		
6. Positions device vertically with tapered end cap facing downward and white fabric facing patient's labia.		
7. Aligns Tapered End Cap with perineum and places device between the labia and against the urethral opening. Ensures absorbent fabric covers the urethral meatus.		
8. Curves the device toward the patient's lower abdomen.		
9. While holding the device in place, removes the adhesive liner and smoothly adheres the adhesive pad over the suprapubic region.		
10. Sets vacuum to low, continuous suction at a minimum of 40mmHg. Always uses minimum amount of suction necessary.		
11. Returns patient's legs to naturally closed position.		
Removal	Completed	Comments
12. Places the patient in the supine position and separates the patient's legs.		

Sage PrimaFit® External Urine Management for the Female Anatomy

Competency skill checklist

13. Gently removes the adhesive pad.		
14. Separates the labia and removes the device.		
15. Assesses skin integrity and removes the device.		
16. Disconnects device from suction tubing and discards device per hospital protocol.		

Employee signature

Observer signature