

Sage PrimoFit™ External Urine Management for the Male Anatomy



Competency skill checklist

Name:
Date:

Discontinue use:

- When independent or assisted toileting is feasible.
- During patient ambulation.

Prior to use	Completed	Comments
1. Performs skin assessment every two hours per hospital protocol.		
2. Refers to manufacturer’s instructions for suction set up.		
Placement	Completed	Comments
3. Sets up suction canister with tubing per hospital protocol.		
4. Places patient in supine position and separates patient’s legs.		
5. Cleans and dries patient’s genital area per hospital protocol and assess skin integrity.		
6. Removes device from package.		
7. Removes the Base Adhesive liner.		
8. Positions the device with adhesive side toward the patient with Tabs facing upward and the opening of the Base Adhesive in line with the patient’s penis.		
9. Holding the Tabs, positions the Base Adhesive up and around the patient’s penis against the skin. To ensure adequate adhesion, applies pressure all the way around the Base Adhesive against the skin.		

Sage PrimoFit™ External Urine Management for the Male Anatomy

Competency skill checklist

10. Attaches suction tubing to the device at the Suction Tubing Adapter. Ensures connection of suction tubing to the device is secure. Confirms there are no kinks in the tubing and that tubing does not run under the patient.		
11. Sets vacuum to low, continuous suction at a minimum of 40mmHg.		
12. Returns patient's legs to a closed position.		
13. Confirms patient is centered on the Glide Sheet.		
Removal	Completed	Comments
14. Places the patient in the supine position and separates patient's legs.		
15. Gently removes the Adhesive Pad from the patient's abdomen.		
16. Removes the patient's penis from the Wicking Chamber.		
17. Gently removes the Base Adhesive by pulling the Tabs down and away from the patient's penis.		
18. Assesses skin integrity and performs perineal care.		
19. Disconnects device from suction tubing and discards device per hospital protocol and local regulations.		

Employee signature

Observer signature