

Sage

2% Chlorhexidine Gluconate Cloth

Competency skill checklist



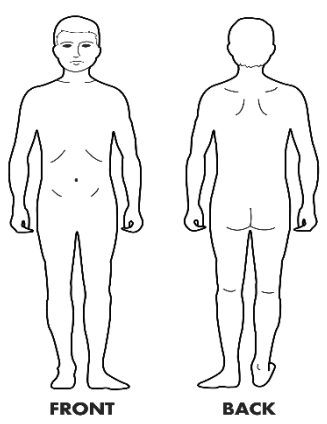
Name:
Date:

Sage's 2% CHG Cloths			
Gathering product:		Completed	Comments
1.	Inspect warmer for expired product.		
2.	Check settings for appropriate temperature.		
3.	Packages are ready for use when the READY indicator is illuminated.		
4.	Gather the appropriate amount of product based on the facility protocol.		
To open package:		Completed	Comments
1.	Holding the top of the package in one hand, lift the flap on the backside of the package with your other hand.		
2.	Grasp flap at top and pull down to tear flap away and expose foam.		
3.	Hold outside of package to present foam and cloths to prep table, avoiding contact between cloths and outside of package to reduce risk of cloth contamination.		
4.	Transfer contents onto prep table, avoiding contact between cloths and outside of package to reduce risk of cloth contamination.		
Warnings:		Completed	Comments
1.	Check with the patient to see if the patient is allergic to chlorhexidine gluconate or any other ingredients in the product. Do not use if they have an allergy.		
2.	Do not use on eyes, ears, and mouth. If contact occurs, rinse with cold water right away and contact a doctor.		

Sage

2% Chlorhexidine Gluconate Cloth

Competency skill checklist

Prepping the patient's skin:		Completed	Comments
1.	<p>Follow the protocol chosen by the facility. Indicate prep area(s) on figure below.</p> <p>Fill in protocol here:</p> <div style="text-align: center; margin: 20px 0;">  </div>		
2.	Discard any unused cloths.		

Employee signature

Observer signature